

1. Locating/downloading update files

- 1.1. Go to <http://www.hibergene.com/distributors/distributor-portal/>
- 1.2. Locate the relevant files in the **Assay Method Files (AMFs)** side bar.
- 1.3. Download the relevant AMF to your laptop/PC (It will download as a 'zip folder').
- 1.4. Located and extract the compressed files to a chosen location on your laptop/PC, e.g. Desktop.

2. Installation of AMFs onto HG SWIFT

- 2.1. Using the USB cable supplied with the instrument, connect the HG Swift to a laptop/PC and turn on the power.
- 2.2. Locate and copy the relevant AMF on your laptop/PC.
- 2.3. Open File Explorer. The attached HG Swift will be visible as an external hard drive, named HGS-0XXX, where XXX is the instrument serial number. Paste the copied files to the location \HGS-0XXX\HGSWIFT\AMF.
- 2.4. Restart the HG Swift. The new assay will now be available for selection from the drop-down menu when starting a new run.